


Q.C.M. Retainer Wire Type W/PLB Changeable



Q.C.M. Retainer Wire W consists of plastic labial bow (PLB) and metal wires, connected by screws.

In case of stains and/or breakages of the plastic bow, just change the PLB leaving the acrylic plate as it is.

Quick Change Method

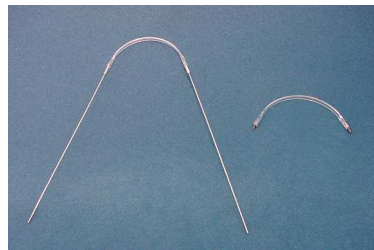

Product specification

The size is 50mm only.


RWL/S W50	5 pieces/package
RWL/S W50	10 pieces/package
PLB	5 pieces/package

How to change plastic labial bow

Stained with tea or plaque?
Accidentally broken?




Loosen its screws and
Remove the damaged labial bow

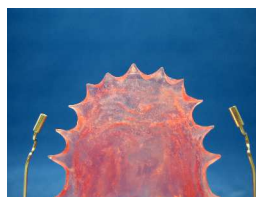

Change a new
labial bow



Completed!

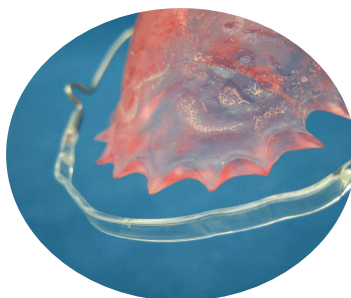

Heat it with a hair dryer and fit it
on a dental arch



Place it on a dental model

